



QUICK GUIDE v.1.0.0

MDS-E100 / MDS-R100 / MDS-T100

MDS-F100 / MDS-A100



I CONTENTS

2 Introduction
Features

3 Usage scenario
Accessories

4 Device overview

6 Specifications

7 Installation

8 Setting

9 Wifi

10 Update

11 Server

How to use

13 RS232 Mode

15 Image Mode



**For more information
please refer to online user manual.**

Introduction

Up to now, ophthalmology examination still highly depends on labor resource to maintain data. Although modern medical devices become more and more accurate, there exists no integrated system to turn data into structured records. It costs time to organize data and type them into HIS, which causes two critical problems, low efficiency and typos.

The MDS is a total solution to solve these two problems. It automatically fetches data directly from device (which eliminate typo) and intelligently transfer data into e-records (which makes efficiency). The MDS also provides integrated record retrieval interface that helps staff realize patient situation in graphical and statistical manner.

FC CE ✓ RoHS

Features



Support various Output-ready medical devices



Suitable for both clinics and hospitals



Flexible patient identification methods



Automatic data transmission



Data security



Lightweight /Deep-coupling HIS integration




Data analysis



Ready for customization


Usage Scenario

01

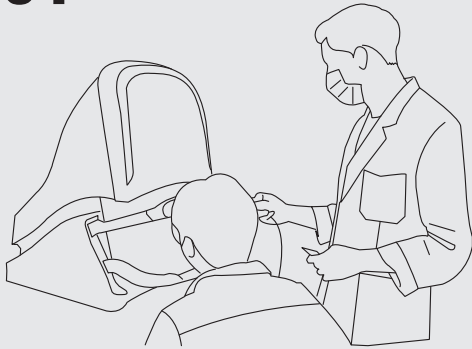

Test

02

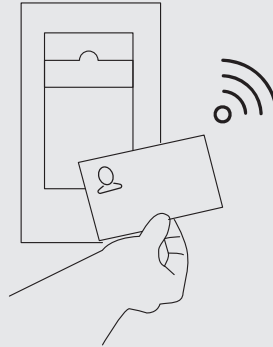


Identify
Scan / Read card

04



Retrieve
Doctors read the e-records


03

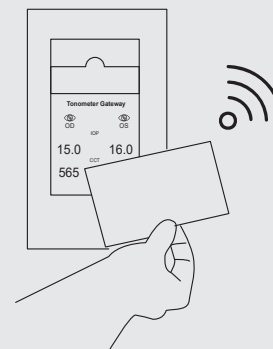


Transfer
Fetch data and upload

Accessories


Power cable

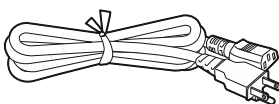

Adapter

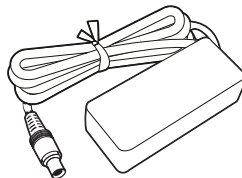


RS232
(Option)

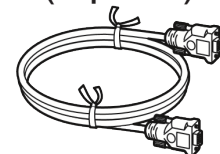


Device Overview

Front View

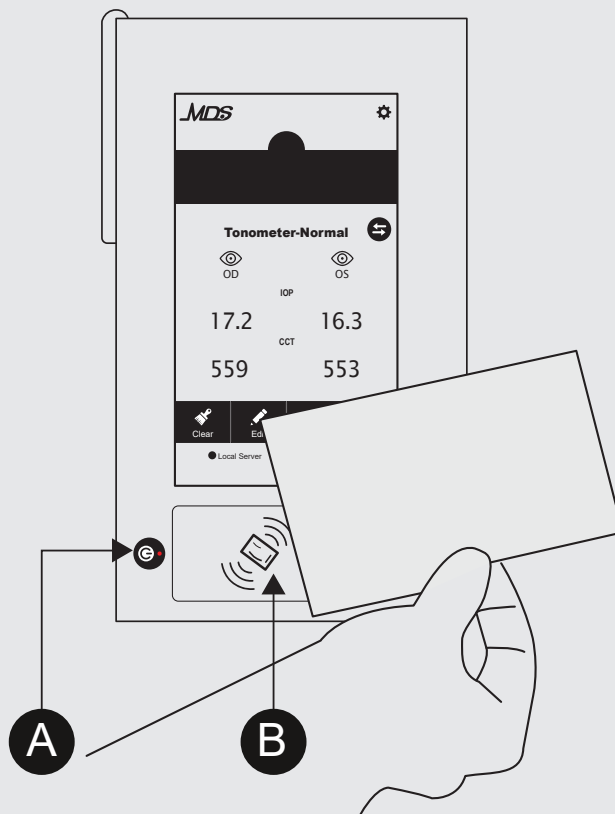


- A Power Key
- B RFID Reader

Bottom View

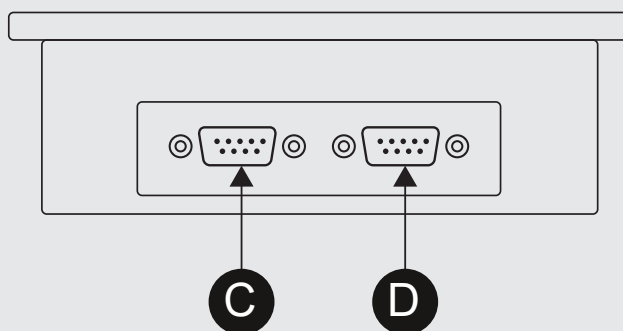

- C COM Port 1
- D COM Port 2

Side View

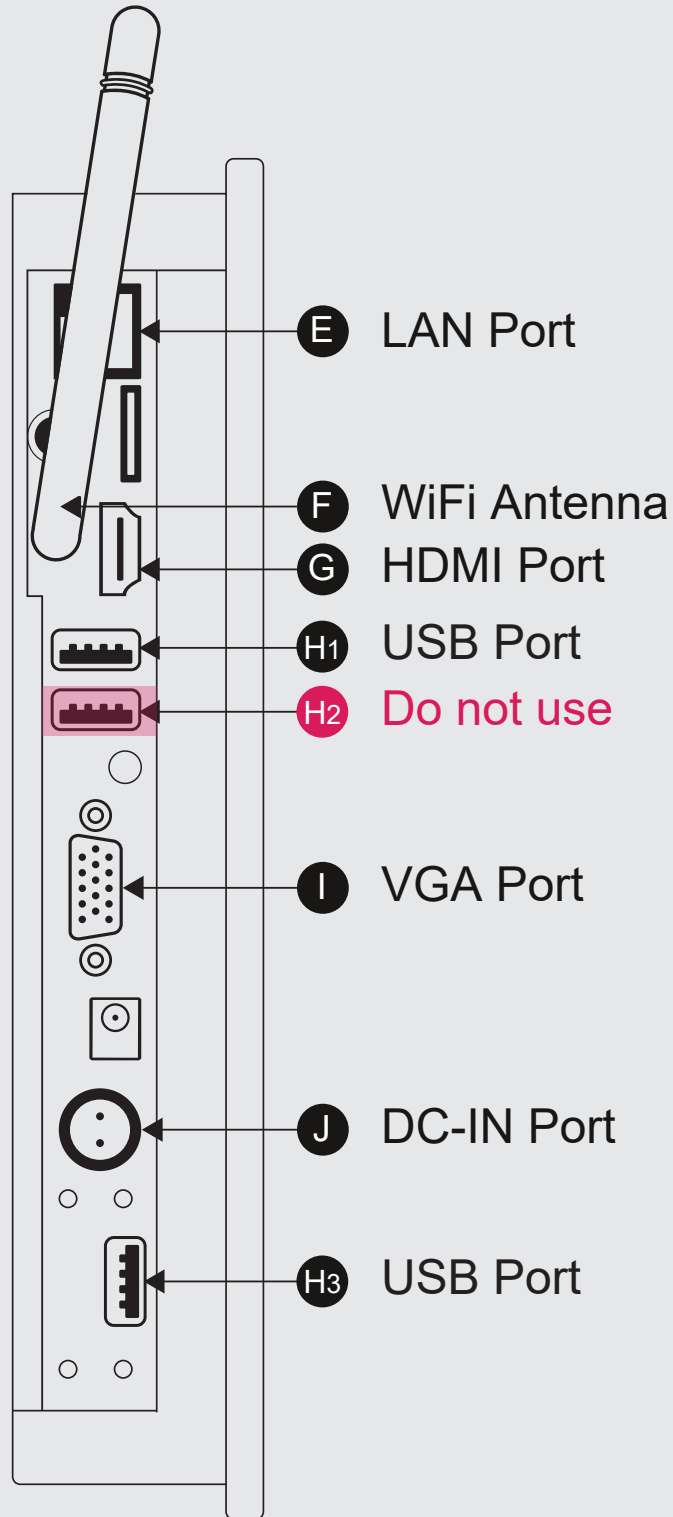

Specifications

Supported devices: see website for detail (www.mdslinx.com)

Dimensions: 250×135×43mm (W x D x H)

Weight: 1.25kg

Screen: 7-inch multi-touch display

Communication: RJ45 ethernet . WI-FI

Patient Identification: Standard Edition: IC card

Advanced Edition: IC card, RFID

Enterprise Edition: IC card, RFID, barcode, QR code

Mounting: VESA 75mmX75mm

Power Supply: 12-24VDC

Power Consumption : 10W

Data Input: RS232, USB, Network

Record Output: Standard Edition: TXT, CSV, JPG files on network drive

Advanced Edition: Network Drive/Database

Enterprise Edition: Network Drive/Database/WEB Service

EMR System: Customized integration with NRE or MDS Data retrieval
EMR system.

MDS Installation

POWER



Take out the MDS, adapter, and power cable from MDS box.



Plug the power cable with the adapter.




Plug the adapter into MDS power connector.




Turn on the power and then MDS will auto start. Long press power key can switch on/off the MDS.


MDS Setting

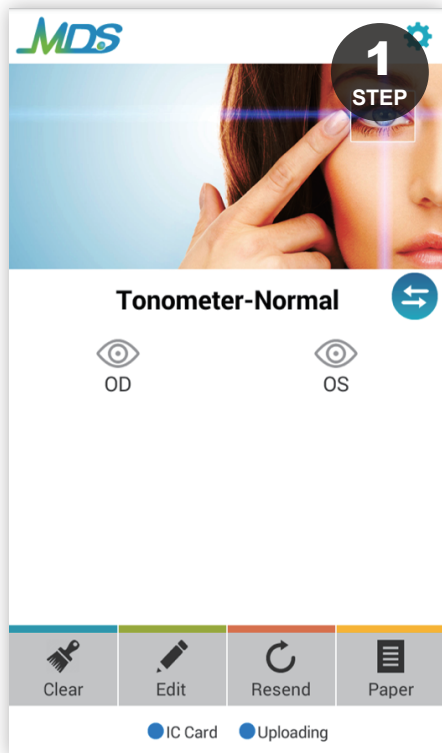

Press the setting icon on the top-right

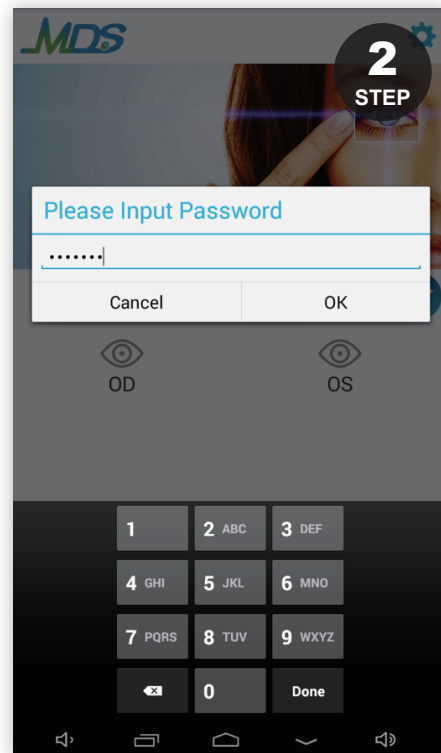


The default password is "5586888"

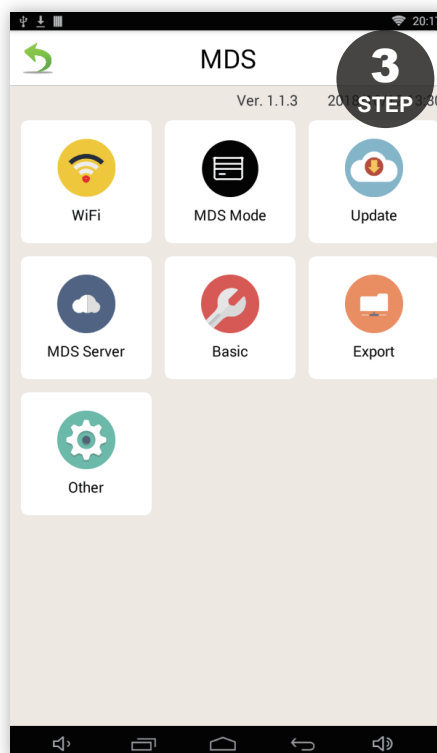


Setting Page


WiFi

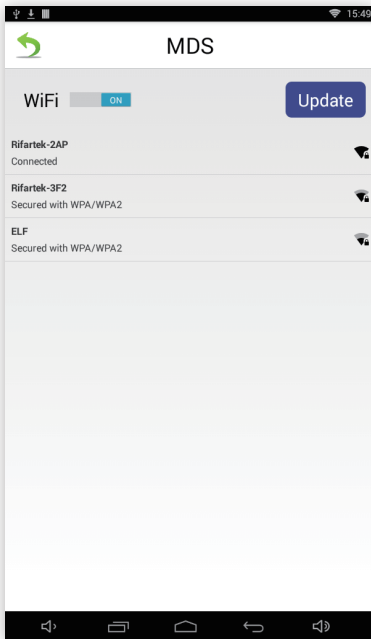


You can press the "Update" button to reload WiFi list in the setting page.

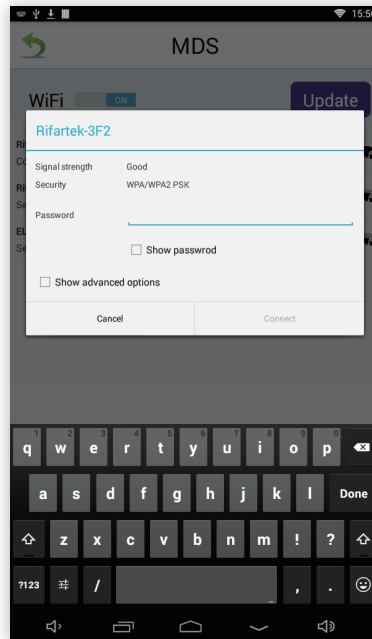


Press the SSID which you want to connect.

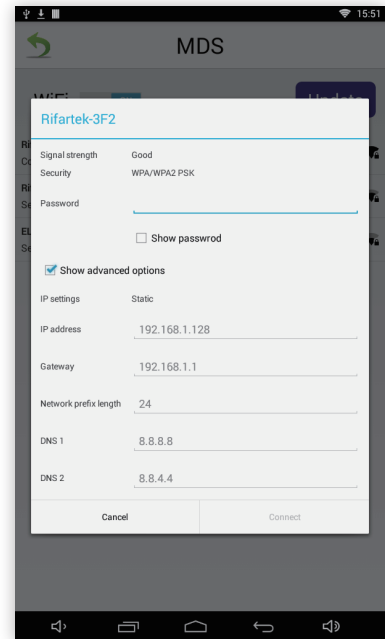


If you want to connect to WiFi by static, press "show advanced options".

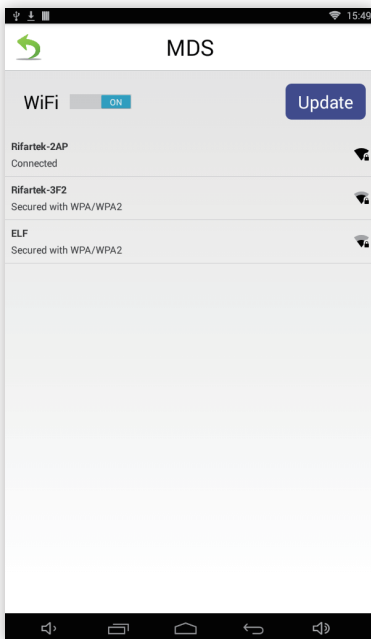

When WiFi connected, the subtitle will show "connected".

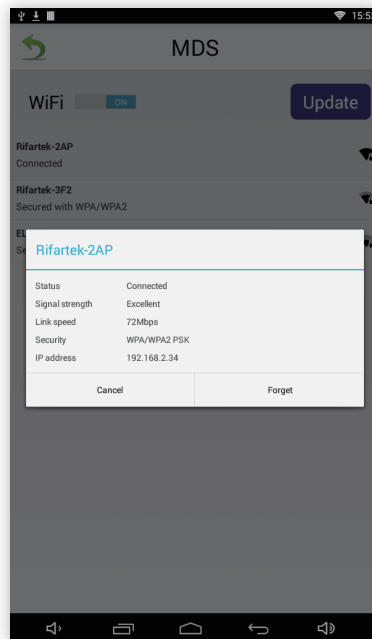


Short press the SSID will pop-up the status' dialog or the selecting dialog.

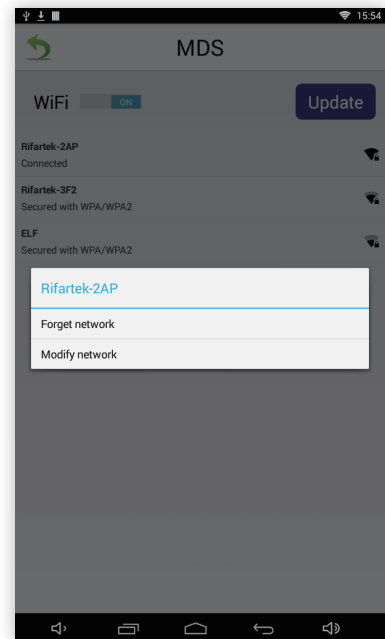

Long press the SSID will pop-up the selecting dialog.

| Update

This page will check online status and notify whether there is a new version.

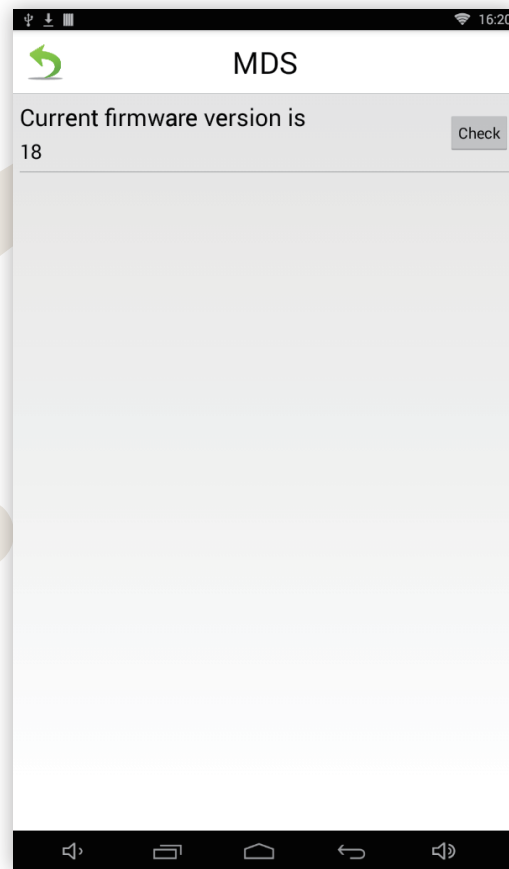

MDS Server

Input the server IP, and it will check the server is alive or not automatically.



Local Server:

- : No WiFi or the server is un-reachable.
- : The server is alive.

The Latest Version:

- : There is a new version for updating, go to "Update Menu" for further information.
- : It's the latest version now.

If you don't need a server, input a random well-formatted IP address with port. And also check the "Not Report Server" checkbox.



How to use

The latest machines support list will show in our website

www.mdslinx.com


RS232 Mode

Choose connected device (Please check online-web and upgrade MDS firmware for more supported device.)

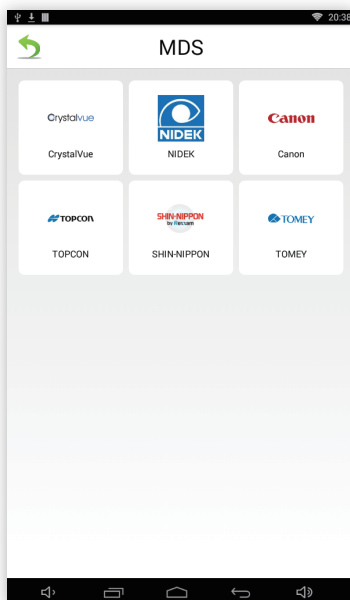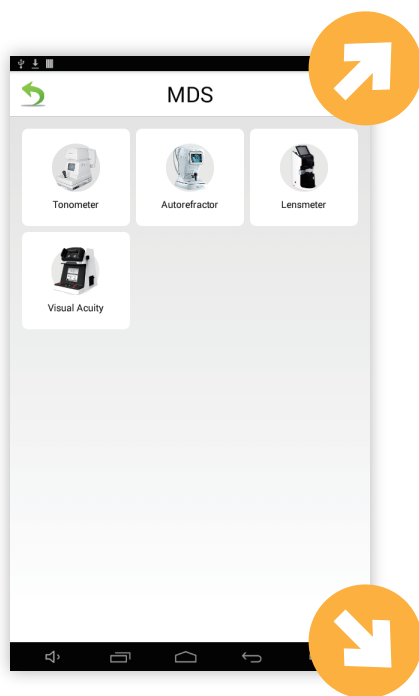


Tonometer

- CRYSTALVUE: TonoVue
- NIDEK: NT-530P, NT-510
- CANON: TX-20P, TX-F
- TOPCON: CT-80
- SHIN-NIPPON: NCT-200
- TOMEY: FT-1000

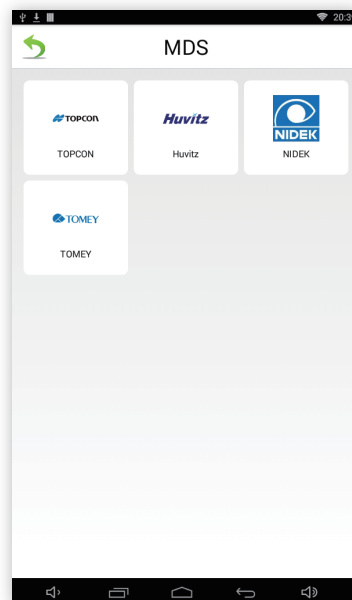


Autorefractor

- TOPCON: KR800, KR 8800, KR8900
- HUVITZ: HRK7000, HRK-8000A
- NIDEK: ARK-1, ARK-1A, ARK-510A
- TOMEY: RC-5000

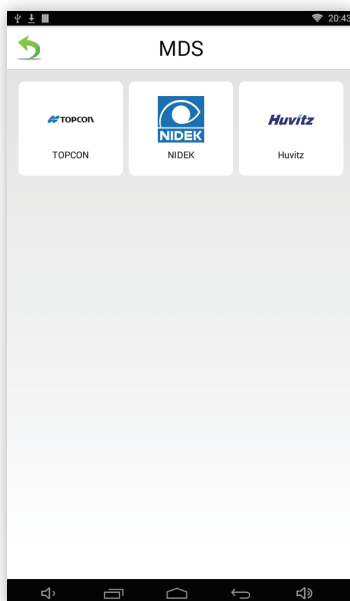

Lensmeter

- TOPCON: CL-100
- NIDEK: LM-7, LM-7P
- HUVITZ: HLM-7000

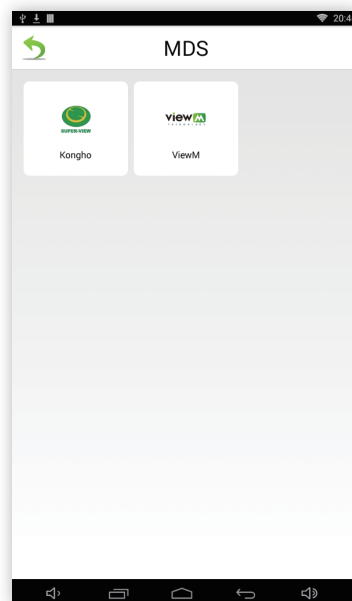

Visual Acuity

- KONGHO: KH525
- VIEWM: VAA-2000

Use Crystalvue Tonometer as demonstration

RS232 Mode



Plug one end of the RS232 Cable in the device.




Plug the other end of the RS232 cable in the left RS232 slot of MDS.



The MDS is enabled to get data from the tonometer

Identify the user profile by one of the following methods.

6 STEP

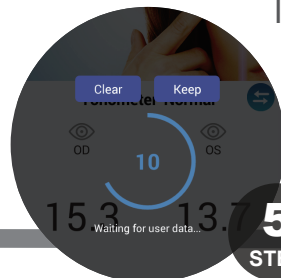

After examination, the result will show on the MDS automatically.

7 STEP



Tap an RFID card on the NFC part of the MDS.

or




Insert an IC card into the IC card reader.

or




Scan a visit code with the Barcode Scanner.

8 STEP

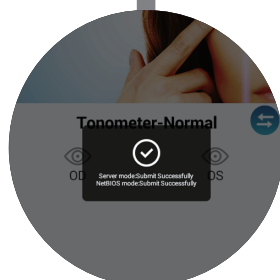


Upload the data to the Server.

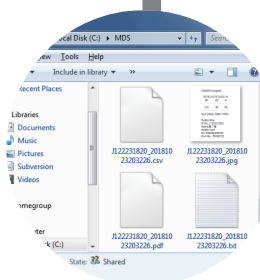


Export the data to a samba path in four types (TXT, PDF, CSV, and JPG).

Image Mode

Choose connected device (Please check online-web and upgrade MDS firmware for more supported device.)

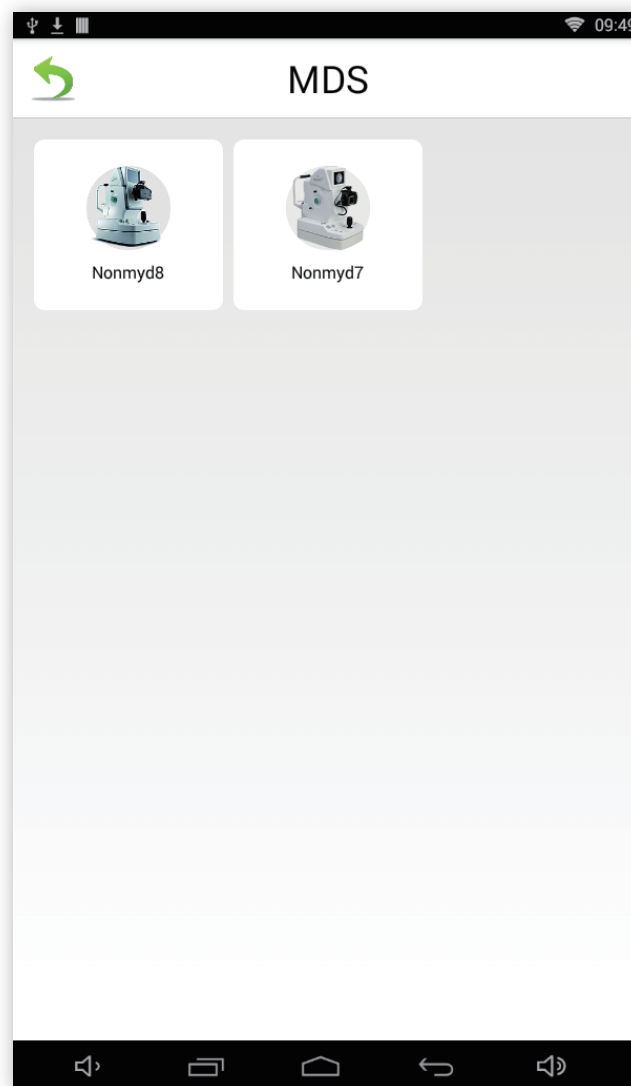

KOWA: Nonmyd 8, Nonmyd7.

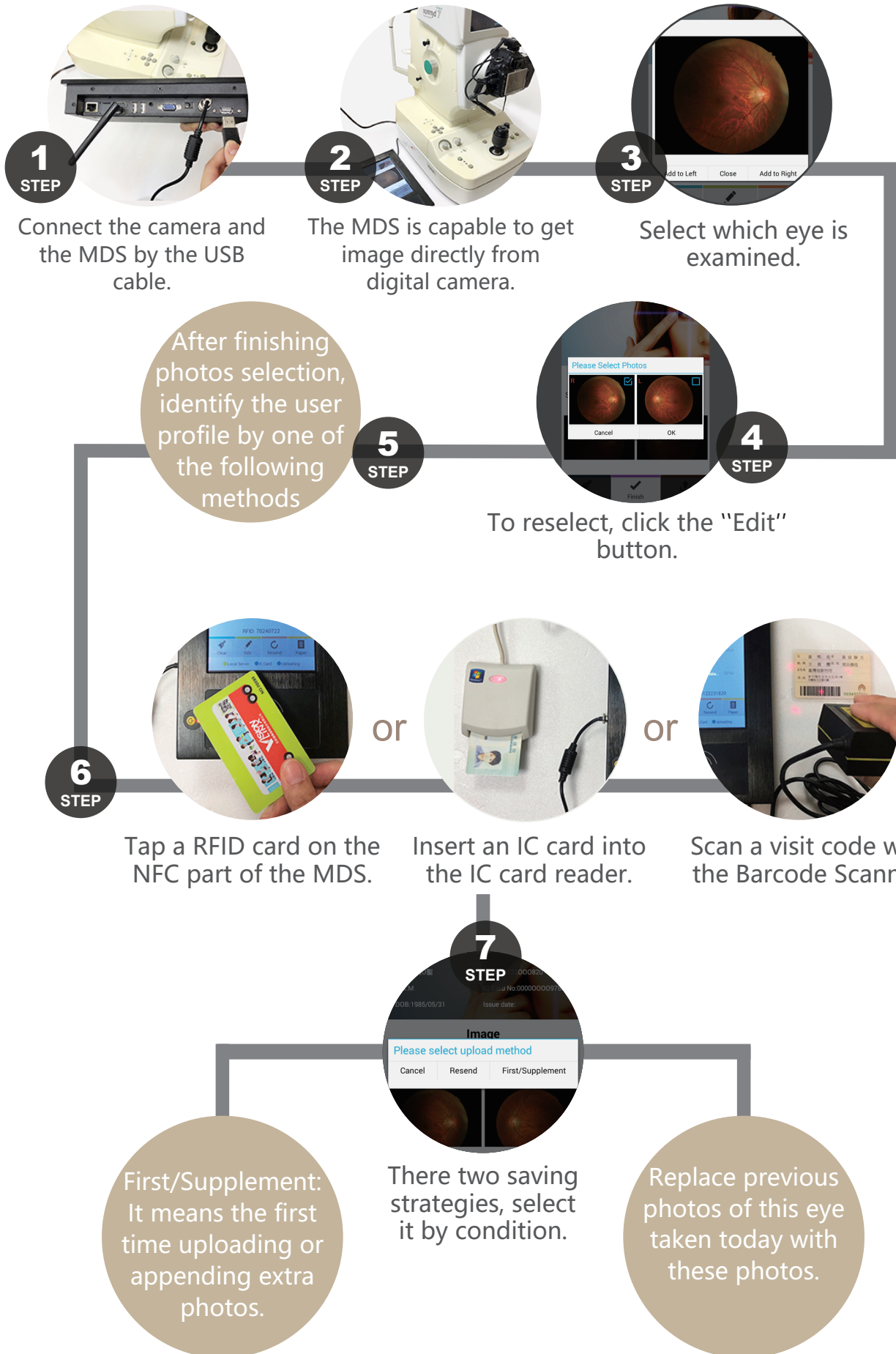



Date of purchase :

Serial number :

Model :

When requesting maintenance, please submit your serial number and model number.

MDSLINX

✉ service@mdslinx.com

🌐 www.mdslinx.com

VISION LINX

✉ service@visionlinx.com

🌐 www.visionlinx.com

Designed by

 **瑞賦科技**
ifartek

☎ +886-3-558-6888 🖨 +886-3-558-9001

✉ service@rifartek.com

🌐 www.rifartek.com